

Roche Cobas 501/ Cobas 311 Instrument Setting

ADA (BSBE)

Method - Enzymatic

Analyze	Calib	Range	Other
Assay / Time / Point (C501)	2 Point Rate	10	32 46 0 0
Assay / Time / Point (C311)	2 Point Rate	10	25 35 0 0
Wavelength (2nd/Pri)	800	546	
Sample Volume	Cassette Configuration		
Norm. 3.3 0.0 0	Code		
Dec. 6.6 0.0 0	Expiration Days 99		
Inc. 1.6 0.0 0	Reagent Volume		
Dilution	R1	120 0	Inactive
	R2	60 0	Bottle Setting
	R3	0 0	
<input checked="" type="radio"/> Water <input type="radio"/> Diluent	0	1	
Linearity Limit	15 %	%	
Prozone Limit	0 32000	0 0	0 0 Inside 0 0
Abs. Limit	32000	Increase	
Cell Detergent	Detergent 1		Stirring Level 2
Stirring Setting	M1	M2	M3
UP	Stirring	LOW	Stirring Stirring Stirring
			SAVE

Analyze	Calib	Range	Other
Calibration Type	Linear		
Point	2		
Span	2		
Weight	0		
Update Type	None	0	0
Auto Calibration	<input checked="" type="radio"/> Timeout Cassette Cance 0 Day		
Changeover	Cassette Cance		
SD limit	999		
Duplicate Limit	99	32000	Abs.
Sensitivity Limit	-99999	99999	
S1 Abs. Limit	-32000	32000	
<input type="checkbox"/> Auto Masking			
			SAVE
QC Violation	<input type="radio"/> QC Violation Method Blank Rule 1 s Control 1 None Control 2 None Control 3 None		

Analyze	Calib	Range	Other
---------	-------	-------	-------

Application Code

Unit U/L

Report Name ADA

Data Mode Active

Automatic Rerun

Technical Kimit

0	176
---	-----

Repeat Limit

-99999	99999
--------	-------

Control Interval Time

Automatic QC On Board Stability

Qualitative

(1)	0		
(2)	0		
(3)	0		
(4)	0		
(5)	0		
(6)	0		

L

0

H

0

I

0

Expected Values

Male

99	Year	0	40
100	Year	-99999	999999
		-99999	999999

Female

99	Year	0	40
100	Year	-99999	999999
		-99999	999999

Default

Sex

Male Female

Range

Range 1 Range 2 Range 3

SAVE

Analyze	Calib	Range	Other
---------	-------	-------	-------

	(1)	(2)	(3)	(4)	(5)	(6)
Calibrator Code	<input style="width: 50px;" type="text"/>	<input style="width: 50px;" type="text"/>	<input style="width: 50px;" type="text"/>	<input style="width: 50px;" type="text"/>	<input style="width: 50px;" type="text"/>	<input style="width: 50px;" type="text"/>
Concentration	0	50				

Rack No. - Pos.

Sample Volume	3.3	3.3				
Diluted S. Volume	0.00	0.00	0.00	0.00	0.00	0.00
Diluent Volume	0.0	0.0	0.0	0.0	0.0	0.0

SAVE

Bottle Setting

Cassette Type A

Bottle

a	R2		
b	R1		
c	R1		

Cance

OK